

STORZ MEDICAL

**EQUIPMENT
CATALOGUE**



YOUR COMPLETE VETERINARY SUPPLY AND SUPPORT SOLUTION





ABOUT EUROVETS

EUROVETS IS YOUR COMPLETE VETERINARY SUPPLY & SUPPORT SOLUTION

Eurovets, established in 2001 has become the Middle East's foremost supplier of some of the world's best known veterinary products to veterinary clinics throughout the UAE.

We represent leading multi-national brands in nutrition, veterinary equipment, medicines and consumables for companion, equine and production animals. In addition to the traditional wholesaling, Eurovets provides warehousing, distribution, marketing and sales resources in order to support the product in the region.

We also offer a host of value-added services such as maintenance and support for all equipment supplied; service-based offerings such as allergy testing; even end-to-end IT solutions for clinics.

Eurovets supports Continued Professional Development, imparting knowledge of the latest developments in the field to veterinary professionals through high-powered seminars.

We pride ourselves on our professional approach and offer vet specialized product advice where required.

In conclusion our mission is to not only deliver excellent products but to be committed to deliver exceptional customer service to all veterinary clinics and pet retailers in the region.

Eurovets YOUR headquarters for veterinary supply and support and a proud distributor of the Storz Medical brand.

STORZ MEDICAL

STORZ MEDICAL AG is an independent partner company of the KARL STORZ Group. The objective pursued by our physicists and engineers is to continuously improve shock wave technology, to develop new system concepts and to identify new therapy indications in close co-operation with leading medical institutes.

With its state-of-the-art technological innovations, STORZ MEDICAL is setting new standards in cardiology, orthopaedics, rheumatology, rehabilitation medicine, aesthetic medicine, dermatology and veterinary medicine.

Shock wave systems developed by STORZ MEDICAL for veterinary applications, among which the DUOLITH® SD1 T-TOP VET »F-SW ultra« and the highly mobile MASTERPULS® MP100 VET »ultra«, offer a variety of convincing benefits such as maximum mobility, low capital expenditure and maintenance costs, excellent performance and vast spectrum of applications and, hence, guaranteed therapy success.



INDEX

PAGE

Shock wave therapy in veterinary medicine	4
MASTERPULS® MP100 VET »ultra«	6
DUOLITH® SD1 T-TOP VET »F-SW ultra«	8
Shock wave applications in horses (examples)	10
Shock wave applications in small animals (examples)	12

Extracorporeal shock wave therapy
IN VETERINARY MEDICINE

STORZ MEDICAL



Shock wave therapy in veterinary medicine

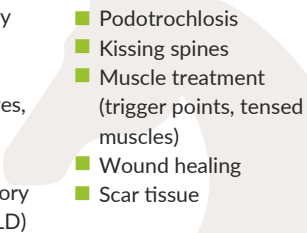
Extracorporeal shock wave therapy (ESWT) was first used for the treatment of orthopaedic disorders in the mid-1990s. Since then, it has become a standard procedure in pain therapy in humans, horses and small animals. Shock wave systems available in the market today use radial pressure waves or focused shock waves. Focused shock wave are ideal for the treatment of deep target areas, whereas pneumatically generated radial pressure waves are used for the treatment of superficial disorders.

What are the effects of shock wave therapy?

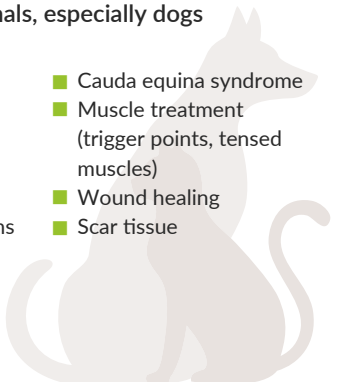
Shock waves have been demonstrated to produce antibacterial and anti-inflammatory effects. Additionally, growth factors have been found to increase in blood vessels, bones and connective tissue. As well as instant analgesic effects observed during the therapy, shock waves primarily have a long-term influence.

Generally speaking, it can be said that the biological effects induced by shock waves produce a time-shifted and sustained response inside the body. However, pain tolerance differs between horses and small animals, and they respond differently to shock wave therapy. As a consequence, accurate diagnostic examination and differential diagnosis are fundamental pillars of successful treatment.

Indications in horses

- 
- Insertional desmopathy
 - Tendinopathy
 - Osteoarthritis
 - Fissures, stress fractures, osteolysis
 - Ligament injuries
 - Degenerative suspensory ligament desmitis (DSL D)
 - Podotrochlosis
 - Kissing spines
 - Muscle treatment (trigger points, tensed muscles)
 - Wound healing
 - Scar tissue

Indications in small animals, especially dogs

- 
- Tendinopathy
 - Osteoarthritis (e.g. spondylarthritis)
 - Hip and elbow dysplasia
 - Fissures, stress fractures
 - Osteochondritis dissecans (OCD)
 - Cauda equina syndrome
 - Muscle treatment (trigger points, tensed muscles)
 - Wound healing
 - Scar tissue

Advantages and benefits of ESWT

- Fast alleviation of pain or even complete pain relief
- Accelerated healing process
- No side effects
- No sedation or anaesthesia required
- Mobile application
- Outpatient treatment
- Reasonable and predictable costs
- No more than 3 to 5 treatments



MASTERPULS® MP100 VET »ultra« – Mobile radial pressure wave therapy

The MASTERPULS® MP100 VET »ultra« radial pressure wave system offers a range of benefits: compact size, excellent reliability, low servicing costs and an extremely silent built-in »Air Power« drive.

Plus unique ergonomic features: All essential controls (buttons and display) are located on the handpiece to ensure uninterrupted and fast handling and application without having to move away from the patient. The extra-long handpiece cable means that the system as such remains in the background.

Treatment parameters developed by experienced therapists for all ESWT indications can be conveniently selected using the handpiece buttons. Alternatively, however, the system can be controlled via an optional touch screen.

»FALCON« radial handpiece



Technical specifications – MASTERPULS® MP100 VET »ultra«

System weight: 10.5 kg

Extra-long handpiece cable (3.50 m)

Pressure: 0.3 – 5.0 bar_{eff}

Frequency: 21 Hz

Optional vibration therapy with V-ACTOR® handpiece: 31 Hz

Optional control via 10" touch screen

Handpiece display



Highlights

- Mobile use (carrying bag or case for transport)
- 1 minute set-up
- Effective and non-tiring handling without having to move away from the patient
- All controls (buttons and display) directly on the handpiece
- Therapeutic effectiveness up to 50 mm penetration depth
- Optional control via 10" touch screen

Carrying bag for transport





DUOLITH® SD1 T-TOP VET »F-SW ultra« – Mobile focused shock wave therapy

With its high-quality workmanship and optimized ergonomics, the design of the new generation of focused shock wave systems with »SEPIA« handpiece is truly impressive. The long handpiece cable is particularly flexible, which ensures non-tiring handling and easier treatment without having to move away from the patient – a crucial benefit in daily practice.

Technical specifications – DUOLITH® SD1 T-TOP VET »F-SW ultra«

System weight: 24.0 kg

Extra-long handpiece cable (2.55 m)

Energy: 0.01 – 0.35 mJ/mm², cable length 2.55 m

Energy (optional): 0.01 – 0.55 mJ/mm², cable length 1.60 m

Optional control via 10" touch screen

Highlights

- Mobile use (carrying case for transport)
- Effective and non-tiring handling without having to move away from the patient
- All controls (buttons and display) directly on the handpiece
- Reduced revision costs thanks to easy change of coil
- Focal zone depth: 0 – 65 mm
- Therapeutic effectiveness: up to 125 mm penetration depth
- Exchangeable stand-offs for precise treatment

»SEPIA« focused handpiece with and without stand-off



Handpiece display



Accessories case and carrying case for transport





Shock wave applications in horses (examples)

Radial pressure wave therapy: tendinopathy



Radial pressure wave therapy: trigger point treatment



Focused shock wave therapy: osteoarthritis (bone spavin)



Focused shock wave therapy: kissing spines



Focused shock wave therapy: podotrochlosis



Focused shock wave therapy: osteolysis





Shock wave applications in small animals (examples)

Radial pressure wave therapy: hip dysplasia



Radial pressure wave therapy: trigger point treatment



Focused shock wave therapy: cauda equina syndrome



Focused shock wave therapy: metatarsal stress fracture



Focused shock wave therapy: knee osteoarthritis



Focused shock wave therapy: elbow dysplasia



NOTES

STORZ MEDICAL

HUMANE TECHNOLOGY – TECHNOLOGY FOR PEOPLE



Urology



Orthopaedics



Cardiology



Aesthetics



Dermatology



Veterinary

Eurovets YOUR complete veterinary Supply and Support SOLUTION.



Copyright © 2021

Eurovets Veterinary Medicines LLC

All rights reserved

CONTACT US:

Tel: +971 4 434 2436

Whatsapp: +971 58 170 1736

E-mail: orders@eurovetworld.com

Address: Warehouse A6, Dubai Science Park, Dubailand,
Al Barsha South, Dubai, United Arab Emirates

For online orders visit:
eurovets.ae



eurovetworld.com